



INTEGRA



CATALOGUE **TOKUYAMA** DENTAL MATERIAL



DENTAL MATERIAL / TOKUYAMA / RESTORATION



CATALOGUE

05 WHY DENTAL MATERIAL TOKUYAMA?

07 TOKUYAMA PALFIQUE LX5

Profile Product PALFIQUE LX5

09 TOKUYAMA PALFIQUE LX5 SYSTEM KIT

Profile Product Palfique LX5 System Kit

11 TOKUYAMA PALFIQUE UNIVERSAL FLOW MEDIUM

Profile Product Palfique Universal Flow Medium

13 TOKUYAMA PALFIQUE UNIVERSAL FLOW SUPERLOW

Profile Product Palfique Universal Flow Superlow

15 TOKUYAMA PALFIQUE BULK FLOW

Profile Product Palfique Bulk Flow

17 TOKUYAMA BRUSH NO.24

Profile Product Brush No. 24

19 TOKUYAMA PALFIQUE ASTERIA

Profile Product Brush No. 24

CATALOGUE

21

TOKUYAMA PALFIQUE ASTERIA ESSENTIAL KIT

Profile Product Palfique Asteria Essential KIT

23

TOKUYAMA PALFIQUE OMNICHROMA

Profile Product Palfique Omnichroma

26

TOKUYAMA PALFIQUE OMNICHROMA BLOCKER

Profile Product Palfique Omnichroma Blocker

28

TOKUYAMA PALFIQUE OMNICHROMA FLOW

Profile Product Palfique Omnichroma Flow

30

TOKUYAMA PALFIQUE OMNICHROMA BLOCKER FLOW

Profile Product Palfique Omnichroma Blocker Flow

32

TOKUYAMA PALFIQUE BOND

Profile Product Palfique Bond

34

TOKUYAMA PALFIQUE UNIVERSAL BOND KIT

Profile Product Palfique Universal Bond Kit

36

TOKUYAMA REBASE II FAST KIT

Profile Product Rebase II Fast Kit

CATALOGUE

39

TOKUYAMA SOFRELINER TOUGH M KIT

Profile Product Sofreliner Tough M Kit

41

TOKUYAMA SHIELD FORCE (PLUS)

Profile Product Shield Force (Plus)

43

CUSTOMER SUPPORT

Monday - Friday : 8am - 5pm. (GMT +7)

Saturday : 8am - 1pm. (GMT + 7)

OVERVIEW Tokuyama

WHY DENTAL MATERIAL?

Tokuyama - A Century Of Caring

When Tokuyama Corporation was originally established as Nihon Soda Kogyo Co., Ltd. in 1918 in Japan, it was with the goal to enhance people's lives. For the following century, Tokuyama held firm to this mission, and used its chemical expertise to improve everyday items from diapers to transition eyeglasses. As a subsidiary of Tokuyama Corporation, Tokuyama Dental operates under the same philosophy, and is committed to the development of top-quality dental products. We ensure our products meet the highest standards of the industry and listen to your needs to optimize the outcome for doctors and patients. We are Innovating Tomorrow's Dentistry, Today™.



Tokuyama Factory Yamaguchi, Japan

Global Manufacturer

With production plants and subsidiaries across the globe and more than 5,400 employees, Tokuyama Corporation is one of the largest and most advanced chemical and silicon manufacturers in the world. Well-known for its research and development, cutting-edge technology and innovative chemistry, Tokuyama Corporation applies a global network of experience and excellence across all its product divisions.



R&D Facility Tsukuba, Japan



Innovation

Innovation is in our DNA. Tokuyama Research and Development is dedicated to creating products that provide more efficient ways to satisfy the needs of dental professionals and their patients.



Excellence

As a Japanese company, we strive for excellence. We are committed to developing the best solutions for the dental industry.



Quality

Developing products that consistently perform at or above clinical expectations is our top priority. Numerous awards and recognitions validate our focus on producing quality products.



Superior Customer Service

We treat our customers like family, and as such, we are determined to offer quick and easy solutions for our customers. Providing superior service is our primary mission.



Social Responsibility

Tokuyama Dental is committed to Corporate Social Responsibility by contributing to economic development, improving the quality of life of our workforce and their families, and benefiting the local community and society at large. Tokuyama Empowers it strives to increase access to dental care for uninsured and underserved populations.

PALFIQUE LX5



RP 395.000

PALFIQUE LX5

Introducing the next generation of ESTELITE Sigma Quick. Delivering lifelike aesthetic restorations, infused with our Supra-Nano spherical fillers, that was engineered by Tokuyama Dental. Tokuyama Dental's composite resin is recognized world-wide, and features outstanding polishability, wide shade matching range and high wear resistance. PALFIQUE LX5 can meet everything in your practice.



Class II



Before



After

Class IV



Before



After

Contents

Available shades:

A1, A2, A3, A3.5 A4, A5, B1, B2, B3, B4, C1, C2, C3
OA1, OA2, OA3, OPA2, BW (Bleach White),
WE (White Enamel), CE (Clear Enamel)

1 syringe (3.8)



PALFIQUE LX5 SYSTEM KIT

RP 3.570.000

PALFIQUE LX5 SYSTEM KIT

Kit Contents

Palfique LX5 (3.8g) of each shade: - A1	1 Syrg
Palfique LX5 (3.8g) of each shade: - A2	1 Syrg
Palfique LX5 (3.8g) of each shade: - A3	1 Syrg
Palfique LX5 (3.8g) of each shade: - A3.5	1 Syrg
Palfique LX5 (3.8g) of each shade: - 0A3	1 Syrg
Palfique LX5 (3.8g) of each shade: - 0PA2	1 Syrg
Estelite Flow Quick (1.8g) of each shade: - A2	1 Syrg
Estelite Flow Quick (1.8g) of each shade: - A3	1 Syrg
Estelite Flow Quick (1.8g) of each shade: - 0A3	1 Syrg
Bottle Palfique Bond Force (5ml)	1 Btl
Dispensing Well	50 pcs
Shade Guide	1 pcs

TOP AWARD DENTAL ADVISOR WINNER



[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

PALFIQUE UNIVERSAL FLOW MEDIUM

RP 390.000

PALFIQUE UNIVERSAL FLOW MEDIUM


Contents

Available 12 shades: A1, A2, A3, A3.5 A4, A5, B3, 0PA2, 0PA3, 0PA4, BW (Bleach White), WE (White Enamel), CE (Clear Enamel)

1 syringe (3.0g/1.8ml) with Dispensing Tip x 10 pcs

PALFIQUE® Universal Flow is the universal flowable composite resin having various features of excellent esthetic, physical-mechanical and handling properties as summarized below:

Flowability

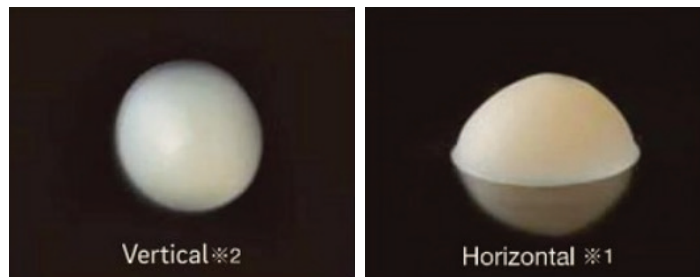

Recommended Cavities



[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

PALFIQUE UNIVERSAL FLOW SUPERLOW

RP 390.000


PALFIQUE UNIVERSAL FLOW SUPERLOW

Contents

Available 6 shades: A1, A2, A3, A3.5 A4, A5

1 syringe (3.0g/1.8ml) with Dispensing Tip x 10 pcs

Flowability

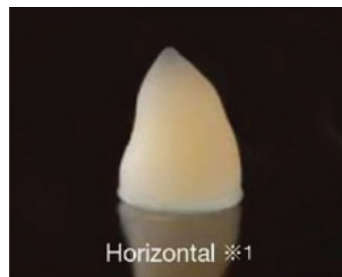

Recommended Cavities



Class I



Class II



Class III



Class IV



[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

PALFIQUE BULK FLOW

RP 400.000

PALFIQUE BULK FLOW

PALFIQUE BULK FLOW is different from the traditional 2mm incremental technique and commercial bulk fill flowable composites requiring a universal cap.

With PALFIQUE BULK FLOW, ONLY one layer up to 4mm is required to complete a restoration. A quick curing time and easy polishing also make the procedure shorter and simpler without sacrificing esthetics.


Contents

Available 5 shades: A1, A2, A3, B1, U

1 syringe (3.0g/1.8ml)

Dispensing Tip x 10 pcs

High Esthetics

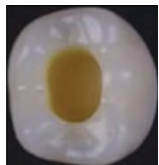

Class I Cavity



Palfique
Bulk Flow (U)



SureFil SDR
flow* (U)



SonicFill* (B1)



Filtek Bulk Fill
Posterior* (B1)

PALFIQUE BULK FLOW shows the highest translucency, which helps light reach the bottom of the restoration and ensure proper curing. PALFIQUE BULK FLOW becomes more opaque than other bulk fill composites when cured, helping the restoration look more natural.



[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

TOKUYAMA BRUSH NO.24

1 Pcs

RP 130.000

1 Pack(5 pcs)

RP 490.000

TOKUYAMA BRUSH NO.24

Tokuyama Brush No. 24 is a double-ended brush used for applying any composite. Clean or disinfect the brushes by using wipes and alcohol, or the brushes can be autoclaved up to 10 times.



Contents

1 Pack : 5 Pcs Brush Tokuyama

1brush (double-different-ended)

PALFIQUE ASTERIA

RP 780.000



PALFIQUE ASTERIA

Palfique Asteria introduces a new simplified 2-step layering concept. This comprehensive system is comprised of minimal shades; just 7 Body shades and 5 Enamel shades cover the entire dental shade range.


Esthetic result is obtained with simple 2-layering concept of Palfique Asteria, because the Body shades replicate chroma and hue and Enamel shades replicate value.

Contents

Body shades: A1B, A2B, A3B, A3.5B, A4B, BL, B3B

Enamel shades: NE, OcE, WE, YE, TE

1 Syringe(2.1 mL/4.0g)

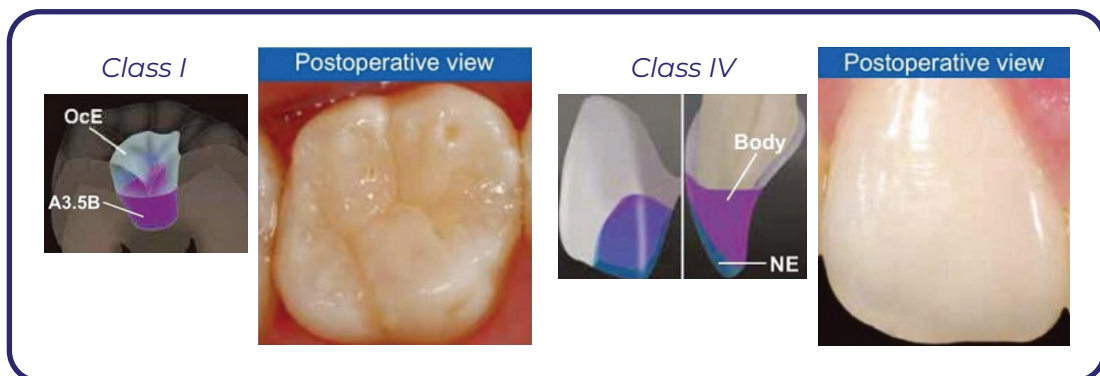



PALFIQUE ASTERIA ESSENTIAL KIT

RP 5.390.000

PALFIQUE ASTERIA ESSENTIAL KIT



The Supra-Nano Spherical filler in PALFIQUE ASTERIA facilitates a very smooth surface with superior gloss that is easily obtained with polishing and is sustained long term.

Contents

Body shades: A1B, A2B, A3B, A3.5B, A4B

Enamel shades: NE, OcE

1 Syringe(2.1 mL/4.0g)

Clinical Guide

PALFIQUE OMNICHROMA

RP 430.000



PALFIQUE OMNICHROMA

PALFIQUE OMNICHROMA is the world's first universal composite that esthetically matches almost all patients with a single shade. Its uniformly sized supra-nano spherical fillers allow PALFIQUE OMNICHROMA to match every shade, a science that we call Smart Chromatic Technology. Its wide color-matching ability eliminates the shade-taking procedure, reducing composite inventory and saving doctors time and money.

Contents

1 Syringe(4 gr/2.2 mL)



Tooth Color

All smiles have something in common, and that is the colors that make up human teeth shades. All shades of teeth express a narrow range of red-to-yellow color. The Munsell Sphere on the right demonstrates the color range of teeth. To match every smile, most composite brands require multiple shades.

That's because most composites today depend on the color of red and yellow dyes added to the resin material to emulate tooth shades. This means that dentists must keep a large inventory of different composites to accommodate a variety of patients.




Smart Chromatic Technology

PALFIQUE OMNICHROMA exhibits the ultimate wide-range color-matching ability thanks to Tokuyama's Smart Chromatic Technology. The Smart Chromatic Technology is achieved by Tokuyama's uniformly sized 260nm spherical fillers included in PALFIQUE OMNICHROMA.


Structural color occurs when different wavelengths of light are amplified or weakened by the structure of a material itself, expressing colors other than what the material may actually be.

PALFIQUE OMNICHROMA


POSTERIOR

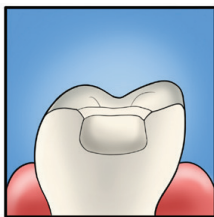


Preparation: Add chamfers to help eliminate margin visibility and aid shade matching.

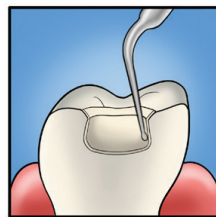


Apply bonding agent.

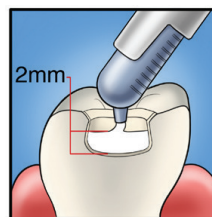

Fill with
PALFIQUE
OMNICHROMA

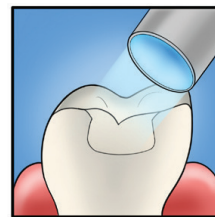


Light cure.
(Curing time
varies depending
on intensity of
curing light)

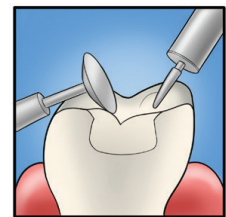


Finish and
polish


ANTERIOR

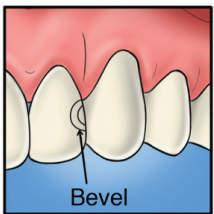


Preparation: Add
bevels to help
eliminate margin
visibility and aid
shade matching.

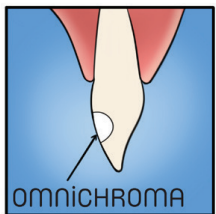

Apply bonding agent
and fill PALFIQUE
OMNICHROMA

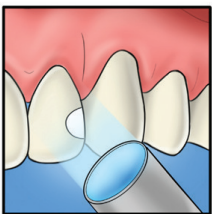

Light cure.
(Curing time
varies depending
on intensity of
curing light)

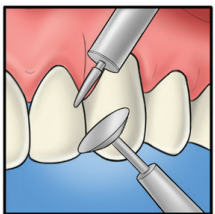

Finish and
polish

PALFIQUE OMNICHROMA BLOCKER

RP 430.000



PALFIQUE OMNICHROMA BLOCKER



In large Class III and IV cases, PALFIQUE OMNICHROMA may experience shade-matching interference due to a lack of surrounding dentition.

PALFIQUE OMNICHROMA BLOCKER is a supplementary material to be used in conjunction with PALFIQUE OMNICHROMA as a lingual layer to prevent


shade-matching interference in these cases. PALFIQUE OMNICHROMA BLOCKER can also be used to mask slight staining, or to reconstruct a highly opaque tooth.

Contents


1 Syringe(4 gr/2.1 mL)

Palfique Omnichroma Blocker


Is used as a lingual layer for large Class III or Class IV restorations with limited surrounding dentition. After applying and curing OMNICHROMA BLOCKER, OMNICHROMA is applied as the second layer.

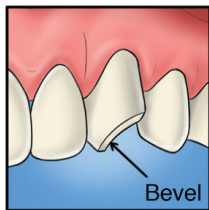


Preparation and apply bonding: Add bevels to help eliminate margin visibility and aid shade matching.

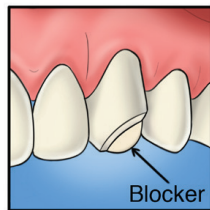


Apply BLOCKER as a lingual layer. Thickness of the lingual layer can vary, but 0.5mm is illustrated as a guide. Light cure.

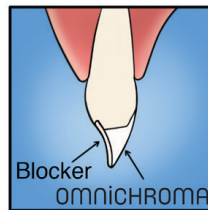

Apply OMNICHROMA as a secondary layer.

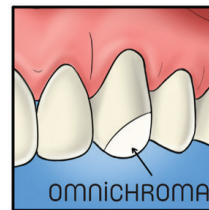

Note that OMNICHROMA is opaque white before curing. It blends naturally after curing.

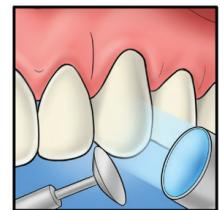

Light cure, finish, and polish. (Curing time varies depending on intensity of curing light.)

Note: PALFIQUE OMNICHROMA appears opaque-white before curing, allowing for higher visibility and easier placement. It becomes the perfect match to the surrounding tooth after curing.



PALFIQUE OMNICHROMA FLOW

RP 430.000



PALFIQUE OMNICHROMA FLOW

OMNICHROMA FLOW has such a wide color-matching range that it is possible to esthetically match the 16 VITA classical shades with just one shade of composite. Therefore, no shade taking is necessary, allowing clinicians to minimize chair time, reduce composite inventory, minimize the wastage of unused composite shades, and reduce reliance on shade-matching procedures. OMNICHROMA FLOW also inherits the features of TOKUYAMA's spherical fillers.

Contents

1 Syringe(3 gr/1.8 mL)

Excellent esthetic properties

- o Unprecedented shade matching
- o High polishability
- o High stain resistance

Excellent physical-mechanical properties

- o Light wear resistance
- o Low polymerization shrinkage

Smart Chromatic Technology

OMNICHROMA FLOW exhibits the ultimate wide-range color-matching ability, covering all 16 VITA classical shades with just one shade of composite, thanks to TOKUYAMA's Smart Chromatic Technology. The Smart Chromatic Technology is achieved through the uniformly sized 260nm supra-nano spherical fillers included in OMNICHROMA FLOW. The scientific background and mechanisms of the Smart Chromatic Technology are detailed in the following sections.


POSTERIOR




Preparation: Add chamfers to help eliminate margin visibility and aid shade matching.




Apply bonding agent.

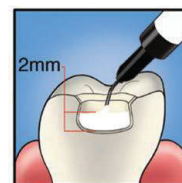

Fill with OMNICHROMA FLOW.

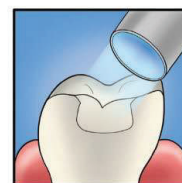


Light-cure.
(Curing time varies depending on intensity of the curing light).

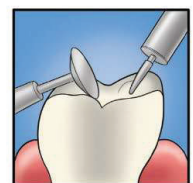


Finish and polish.


ANTERIOR

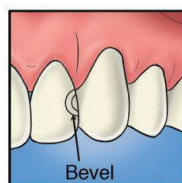


Preparation: Add bevels to help eliminate margin visibility and aid shade matching.

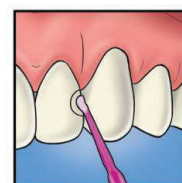


Apply bonding agent.

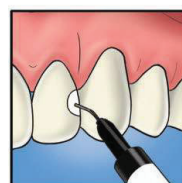

Fill with OMNICHROMA FLOW.

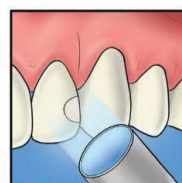

Light-cure.
(Curing time varies depending on intensity of the curing light).

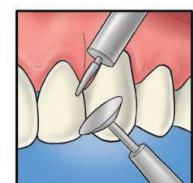

Finish and polish.

PALFIQUE OMNICHROMA BLOCKER FLOW

RP 430.000




PALFIQUE OMNICHROMA BLOCKER FLOW

In large Class III and IV cases with little tooth structure and lack of surrounding dentition, shade-matching interference may occur if only OMNICHROMA FLOW is used. Utilizing OMNICHROMA BLOCKER FLOW overcomes this limitation by working as a supplementary material/ lingual layer which reduces shade-matching interference and provides the perfect color match.


Contents

1 Syringe(3 gr/1.8 mL)

OMNICHROMA BLOCKER FLOW can also be used in cases requiring to mask slight staining (e.g. previously treated with amalgam) or to reconstruct a highly opaque tooth.

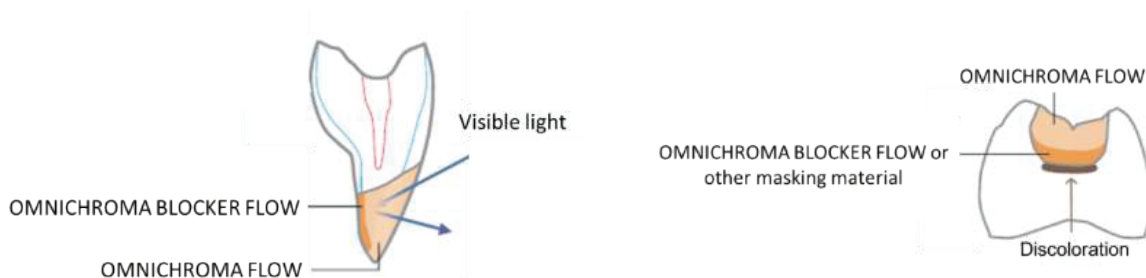

POSTERIOR

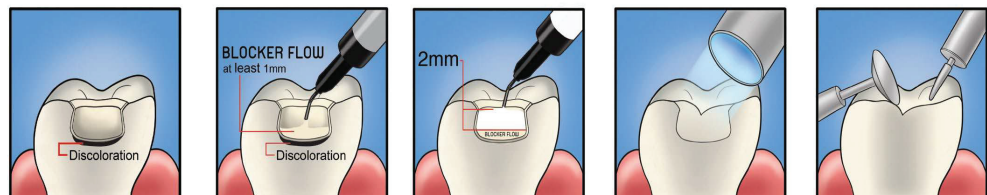

Prepare and apply bonding. Add a bevel to help eliminate margin visibility and aid in shade matching.


Apply BLOCKER FLOW as a base layer to mask discoloration. Thickness can vary, but 1mm is illustrated as a guide. Light-cure.

Apply OMNICHROMA FLOW in 2mm increments.

Light-cure. (Curing time varies depending on intensity of the curing light.)

Finish and polish.

ANTERIOR

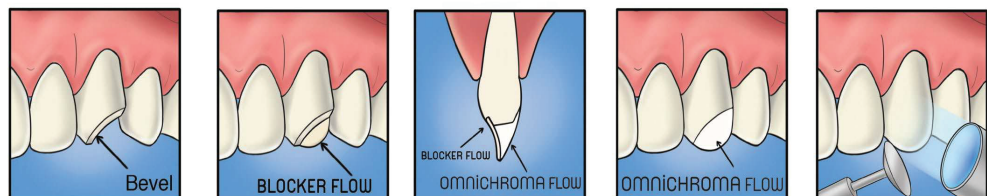

Prepare and apply bonding. Add chamfers or bevels to help eliminate margins and aid shade matching.

Apply BLOCKER FLOW as a lingual layer. Thickness can vary, but 0.5mm is illustrated as a guide. Light-cure.

Apply OMNICHROMA FLOW as a secondary layer.

Light-cure, finish, and polish.

[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

PALFIQUE BOND 5 ML

RP 748.000

DENTAL MATERIAL / TOKUYAMA / RESTORATION

PALFIQUE BOND 5 ML



The 3D SR monomer, originally developed by Tokuyama Dental, creates integrity and a uniform bonding layer. PALFIQUE Bond employs the 3D SR monomer technology to provide a strong and durable adhesive strength, as well as less technique sensitivity and post-operative sensitivity. This is the advantage of a one component, self-etching bonding system which employs the 3D SR monomer. PALFIQUE Bond offers your practice esthetic and reliable bonding.

Feature

Short 10 second application time Store at room temperature (0~25°C) for up to 2 years Short 5 second mild air dry 5 minute working time under a light blocking plate

Contents

1 Bottle(5 ml)

PALFIQUE UNIVERSAL BOND KIT

RP 1.090.000



PALFIQUE UNIVERSAL BOND KIT



PALFIQUE UNIVERSAL BOND is a adhesive system for both direct and indirect restorations that can be used with Self-etch, Selective-etch and Total-etch techniques. As a universal adhesive, PALFIQUE UNIVERSAL BOND has been designed to be fully compatible with light-cured, self-cured and dual-cured composite materials.



PALFIQUE UNIVERSAL BOND also improves the bond strength of polymerizable resin material to all indirect restorative materials

Kit Contents

PALFIQUE UNIVERSAL BOND A / 5mL x 1

PALFIQUE UNIVERSAL BOND B / 5mL x 1

Disposable Applicator (Fine) x 25

Mixing Well x 1

Disposable Mixing Well x 5

REBASE II FAST KIT LIGHTPINK

RP 948.000



REBASE II FAST KIT LIGHTPINK

TOKUYAMA® REBASE II

is a chairside hard denture reline material.

Without compromising any benefits from its predecessor Tokuso® Rebase, Tokuyama® Rebase II offers better polishability, lower heat generation for in-mouth curing higher mechanical strengths, improved handling and less odor/taste. It cures in 4min 30sec intraorally.

Tokuyama® Rebase II is indicated for hard denture relining and partial denture repairs.


Kit Contents

Palfique LX5 TOKUYAMA® REBASE II Powder	1 Pcs
TOKUYAMA® REBASE II Liquid/50mL	1 Pcs
Adhesive/15mL	1 Pcs
Measuring cup (for TOKUYAMA® REBASE II Powder)	1 Pcs
Dropper (for TOKUYAMA® REBASE II Liquid)	1 Pcs
Rubber Cup (for mixing TOKUYAMA® REBASE II Powder and Liquid)	1 Pcs
Spatula (for spreading TOKUYAMA® REBASE II on denture)	1 Pcs
Plastic Cup (for Adhesive)	2 Pcs
Brush (for Adhesive)	2 Pcs
Tokuso® Resin Hardener II	1 Pcs
Spoon (for Tokuso® Resin Hardener II)	1 Pcs

Feature

- One-appointment procedure
- Saving lab costs & time
- In-mouth curing
- Reduced heat & chemical irritation
- Excellent fit
- Adequate working time
- Easy polishing

REBASE II FAST KIT

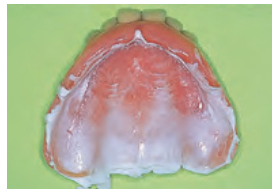

Checkin the adaptation



Removal of the denture surface



Application Of Adhesive




Application of the resin



Insertion of the denture removal of excess resin



Muscle treatment

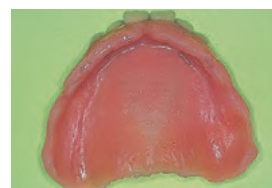


Withdrawal



Immersion in Resin Hardener II Solution



Modifiying and Polishing

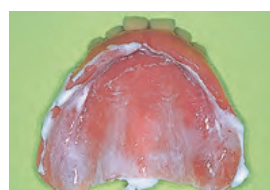

Checking the adaptation

SOFRELINER TOUGH M KIT

RP 1.960.000



SOFRELINER TOUGH M KIT

SOFRELINER TOUGH M(Medium) is an addition-cured silicone chairside soft lining material for dentures. With SOFRELINER TOUGH M(Medium) you can expect the relined denture surface to remain soft, smooth with less-discoloration and less-odor, taste, or chemical irritations. It offers a high tear resistance, high stain resistance and outstanding adhesion to the denture base. With two types of softness (Shore-A hardness),

SOFRELINER TOUGH M(Medium) strikes the optional balance for the clinician who requires a range of softness and durability. SOFRELINER TOUGH M(Medium) to be used when higher durability is the primary concern.

The Silicone Remover (sold separately) is now available, which removes silicone denture liner from the acrylic denture easily.



Kit Contents

SOFRELINER TOUGH M

Paste Base(27g), Catalyst paste (27g)	1 Pcs
Primer(10mL)	1 Pcs
TD II Mixing Tips XS	1 Pcs
Shape adjustment point (White)	1 Pcs
Finishing Point (Brown)	1 Pcs
Spatula	1 Pcs
Drip-cap for primer	1 Pcs
Plastic Cup	2 Pcs
Brush No. 5	2 Pcs

Feature

- Pain caused by sharp ridges of alveolar bone and atrophy of mucosa
- Poor retention caused by extreme ridge atrophy
- Major undercuts or maxillary/mandibular tori
- Retention of over dentures before the final attachment is placed
- Lasting surface smoothness

Indications

- Excellent Durability
- Strong adhesion to the denture
- High tear resistance
- Less-discoloration
- Lasting surface smoothness
- Easy auto-dispensing system

[CATALOGUE](#)[OVERVIEW](#)[LX5](#)[LX5 KIT](#)[FLOW MEDIUM](#)[SUPER FLOW](#)[BULK FLOW](#)[BRUSH NO.24](#)[ASTERIA](#)[ASTERIA KIT](#)[OMNICHROMA](#)[BLOCKER](#)[OMNI. FLOW](#)[BLOCKER FLOW](#)[BOND](#)[BOND KIT](#)[REBASE II](#)[SOFRELINER](#)[SHIELD FORCE](#)

SHIELD FORCE (PLUS) 3ML

RP 798.000

DENTAL MATERIAL / TOKUYAMA / RESTORATION

SHIELD FORCE (PLUS) 3ML



TOKUYAMA® SHIELD FORCE PLUS is a one-component, single application, light-cured protective sealant designed for the treatment of hypersensitive dentin. SHIELD FORCE PLUS effectively infiltrates and seals exposed dentinal tubules to inhibit the flow of fluid through the tubules which is the cause of sensitivity.

At the same time the durable coat of cured SHIELD FORCE PLUS helps prevent loss of dentin due to abrasion and/or erosion of exposed cervical dentin.

Contents


1 Bottle (3 ml)

Indications

- Treatment of hypersensitive dentin
- Reduction of abrasion and erosion of exposed cervical dentin
- Alleviation and/or prevention of tooth sensitivity after tooth preparation for direct and indirect restorations

Benefits


- Quick and effective treatment of hypersensitive dentin
- Durable coating on exposed dentinal tubules
- Superior sealing of prepared tooth
- Simple procedure
- Thin film thickness

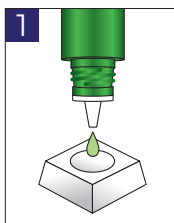


Dispense



Apply SHIELD FORCE PLUS,
then leave for 10+sec

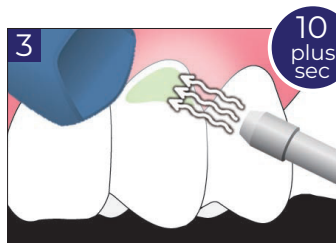

Apply weak air (5sec)
Strong air (5+sec)

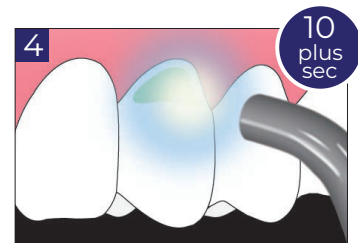

Light cure for 10+sec

CONTACT US

Together, We Craft Beautiful Smiles
Your Clients, Our Exceptional Dental Solutions

We value your feedback and are here to assist you. Please don't hesitate to get in touch with us for any inquiries, suggestions, or concerns. We're committed to providing you with excellent service.



INTEGRA BEST DEAL
0877-4221-8379



INSTAGRAM
@INTEGRA.MEDITAMA



SALES MANAGER
0812-2047-1511



EMAIL
FEBRIAN@INTEGRAMEDITAMA.CO.ID

